

Intermittent Infusion Module

In-service

Priming and Loading the IV Administration Set

- Prime IV Administration Set by gravity and engage roller clamps
- Turn Pump ON -----POST----- will display for 14 seconds as the self-test is completed
- Lift Pump Door Handle to the door
- Load Tubing into the tubing guides from the top down through the Air-In –Line Sensor at the bottom.
- Align Tubing OVER the Anti-Free-Flow clamp, do NOT push tubing into this clamp, the pump door will engage this mechanism.
- Close Pump Door by pushing down on the pump door handle
- Open IV Set roller clamps and VERIFY there is NO FLOW in the IV Set Drip Chamber

Setting up an Intermittent Infusion

- **In the Intermittent mode the pump will infuse for a Period Run Time, followed by an Idle Period Time. The Idle time runs at a KVO. The SUM of the RUN and IDLE time is called the CYCLE Time.**

The Pump Will Display:

- Resume Infusion
- New Infusion (highlighted)
- OK (Soft Key)

Use the Down Arrow Soft Key Choose New Infusion. Confirm with the OK Soft Key (...).

- Use Down arrow key to Highlight:
- **5. Intermittent**
- Use the SELECT Soft Key (...) to select this mode.
- Enter Total VTBI by using the Up and Down Numeric Data Entry Keys that correspond with the correct numeric value.
- Use down Arrow Soft Key to highlight : Cycle Time
- Enter the correct value using the UP and Down Numeric Data Entry Keys.
- Press Period Soft Key
- Enter Period VTBI
- Enter Period Time (Must be between 10 minutes and 23 hours)
- Press RUN to start.

Example: If you want to deliver a MULTI DOSE IV BAG Intermittently (total VTBI 200mL), over SIX HOURS (Cycle Time), Each dose is 50 mL's (Period VTBI) and the run time for each dose is 10 minutes (Period Time), The Idle Time for each Cycle is 5 hours and 50 minutes. (At KVO)

Total VTBI: 500mL

Cycle Time: 06:00

Period VTBI: 50mL

Period Time: 10 Minutes

Unload IV Administration Set

- Once infused is stopped, engage IV set clamps.
- Open pump door by lifting up on the door handle.
- SLOWLY Unload IV set from BOTTOM to Top.
- Close pump door.
- Press and hold OFF for 2 seconds.