

## Building a Protocol Infusion Module

### In-service

#### Priming and Loading the IV Administration Set

- Prime IV Administration Set by gravity and engage roller clamps
- Turn Pump ON -----POST----- will display for 14 seconds as the self-test is completed
- Lift Pump Door Handle to the door
- Load Tubing into the tubing guides from the top down through the Air-In –Line Sensor at the bottom.
- Align Tubing OVER the Anti-Free-Flow clamp, do NOT push tubing into this clamp, the pump door will engage this mechanism.
- Close Pump Door by pushing down on the pump door handle
- Open IV Set roller clamps and VERIFY there is NO FLOW in the IV Set Drip Chamber

#### Saving Infusion Parameters as a Protocol Infusion

The Pump Will Display:

- Resume Infusion
- New Infusion (highlighted)
- OK (Soft Key)

Use the Down Arrow Soft Key Choose New Infusion. Confirm with the OK Soft Key (...).

- Use Down arrow key to Highlight:
- **3. Ten Step Mode**
- Use the SELECT Soft Key (...) to select this mode.
- Enter Primary Rate by using the Up and Down Numeric Data Entry Keys.
- Enter Primary Volume.
- Use down arrow key to proceed to step 2.
- Enter Primary Rate by using the Up and Down Numeric Data Entry Keys.
- Enter Primary Volume.
- Enter Rate and Volume for each step of the Protocol you wish to build.
- Press the Config. Soft Key (...)
- Arrow down to Number **8. Biomed Options**
- Press Select Soft Key (...)
- Arrow down to **8.6 Save As Protocol**
- Press Select Soft Key (...)
- Select 1

- With 10's key, arrow through the alphabet to name your protocol. The UP and DOWN arrow Soft Keys will give you a space. The One's key will give you numbers and a blank space.
- Once you have named your protocol press the SAVE Soft Key.
- Press Quit getting back to the Mode Screen.
- Choose Number **7. Protocols**
- Press Select, and your new protocol will be listed. Press Select again to choose it.
- Press Run to start. (Yellow Key)

## **Unload IV Administration Set**

- Once infused is stopped, engage IV set clamps.
- Open pump door by lifting up on the door handle.
- SLOWLY Unload IV set from BOTTOM to Top.
- Close pump door.
- Press and hold OFF for 2 seconds.