

ZYNO MEDICAL

TPN Module

In-service

Priming and Loading the IV Administration Set

- Prime IV Administration Set by gravity and engage roller clamps
- Turn Pump ON -----POST----- will display for 14 seconds as the self-test is completed
- Lift Pump Door Handle to the door
- Load Tubing into the tubing guides from the top down through the Air-In –Line Sensor at the bottom.
- Align Tubing OVER the Anti-Free-Flow clamp, do NOT push tubing into this clamp, the pump door will engage this mechanism.
- Close Pump Door by pushing down on the pump door handle
- Open IV Set roller clamps and VERIFY there is NO FLOW in the IV Set Drip Chamber

Setting up TPN Infusion

The Pump Will Display:

- Resume Infusion
- New Infusion (highlighted)
- OK (Soft Key)

Use the Down Arrow Soft Key Choose New Infusion. Confirm with the OK Soft Key (...).

- Use Down arrow Soft Key to Highlight:
- **4. TPN Mode**
- Use the SELECT Soft Key (...) to select this mode.
- Enter Time by using the Up and Down Numeric Data Entry Keys that correspond with the correct numeric value.
- Use down Arrow Soft Key to highlight : VTBI
- Enter the correct value using the UP and Down Numeric Data Entry Keys.
- Press run to start **or** use:

RAMP Feature

- Press RAMP Soft Key (-)
- Enter Ramp UP Time using Numeric Data Entry Keys
- Arrow down to enter Ramp Down Time using Numeric Data Entry Keys
- Press RUN to start. (Yellow Key)

Unload IV Administration Set

- Once infused is stopped, engage IV set clamps.
- Open pump door by lifting up on the door handle.
- SLOWLY Unload IV set from BOTTOM to Top.
- Close pump door.
- Press and hold OFF for 2 seconds.