

ZYNO MEDICAL

Time/Volume Module

In-service

Priming and Loading the IV Administration Set

- Prime IV Administration Set by gravity and engage roller clamps
- Turn Pump ON -----POST----- will display for 14 seconds as the self-test is completed
- Lift Pump Door Handle to the door
- Load Tubing into the tubing guides from the top down through the Air-In –Line Sensor at the bottom.
- Align Tubing OVER the Anti-Free-Flow clamp, do NOT push tubing into this clamp, the pump door will engage this mechanism.
- Close Pump Door by pushing down on the pump door handle
- Open IV Set roller clamps and VERIFY there is NO FLOW in the IV Set Drip Chamber

Setting up a Primary Infusion

The Pump Will Display:

- Resume Infusion
- New Infusion (highlighted)
- OK (Soft Key)

Use the Down Arrow Soft Key Choose New Infusion. Confirm with the OK Soft Key (...).

Use the Up and Down Arrow Soft Keys to Select Mode.

- Use Down arrow key to Highlight:
- **2. Continuous Mode Time/Volume**
- Use the SELECT Soft Key (...) to select this mode.
- Enter Primary Time by using the Up and Down Numeric Hard keys.
- Use down Arrow Soft Key to highlight : Primary VTBI
- Enter the correct value using the UP and Down Numeric Hard keys.
- Press Run to start. (Yellow Key)

Set Up Secondary Infusion

- Press Run/Stop to stop Primary Infusion
- Press PROG Soft Key (-)
- Press Select Soft Key (...)
- Press SEC Soft Key (-)
- Enter Time

- Enter Volume
- Press RUN to start
- Pump will alarm when secondary volume complete and automatically switch over to Primary programming.

Unload IV Administration Set

- Once infused is stopped, engage IV set clamps.
- Open pump door by lifting up on the door handle.
- SLOWLY Unload IV set from BOTTOM to Top.
- Close pump door.
- Press and hold OFF for 2 seconds.